

# BrainScope Headset Application

## Skin Prep Instructions

### Step 1: Clean Skin


- Clean full forehead, temples and front earlobes using provided alcohol wipes
- Remove all dirt, oil, sunscreen and make-up

**NOTE:** Heavily soiled skin may need additional cleaning with additional alcohol wipes



### Step 2: Dispense Electrode Gel\*


- Dispense ¾ inch line of the gel on top third of white skin prep pad
- Fold pad in half and rub to dispense gel evenly

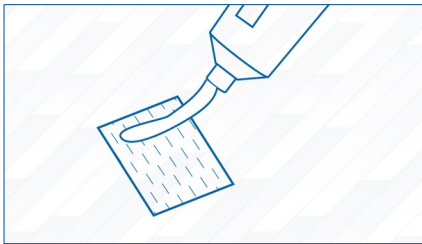

**WARNING:** Do not use gel on patients with known history of skin allergies to parabens

### Step 3: Apply Gel to Skin

- Hold pad between two fingers and thumb, push away hair
- Swipe forehead to evenly spread a thin layer of gel:
  - Center below hairline to bridge of nose
  - Across forehead above the eyebrows to form an inverted “T”
  - Temples from above eyebrow to bottom corner of eye
  - Front of ear lobes (pull lobe to create flat surface)
- Examine pad after each application area, dispense additional gel if needed
- Proceed with headset application

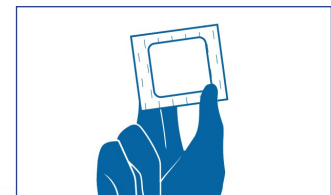

#### NOTES:

- Ensure a visible layer of gel on skin in area of each electrode
- Avoid a thick layer of gel, use pad to smooth
- Do not use pressure
- Keep clear of hair
- If any unacceptable (red) impedances are found, re-prep by adding a small amount of gel

\* If gel is not available, refer to section 3.5 in the user manual for skin preparation via exfoliation.